



Cleanisept Wipes

Disinfection Wipe



Disinfection wipe for non-invasive medical devices and areas that are near the patient and hygienically critical. Contains 100 pcs.

Application

Used for cleaning and disinfection of medical equipment and surfaces (MDD Class IIa). The wipes may also be used for disinfection of sensors of ultrasound equipment. Approved for disinfection of ultrasound heads manufactured by Siemens, Philips, GE, Shimadzu, SonoAce, Aloka, Toshiba, Esaote, Zonare, Hitachi, Sonosite and Samsung Medison, Within food industry and professional kitchens the wipes are used for disinfecting devices used in self monitoring (for example thermometers).

For professional use.

Product properties

The product is microbicidally effective (bacteria, fungi: EN 13697 EN 13697, EN 13624, EN 13727, EN 1650, EN 1276, EN 1275, EN 1040, viruses: EN 14476 (Vaccinia, Polyoma), also EN 16615 wipe test). The disinfecting media is neutral by pH and suitable also for sensitive materials such as various plastics. The product is dermatologically tested. Wipes are made of an anti-lint fabric material.

Package contains 100 pieces of single use fibre fabric wipes, size 14 x 20 cm.

Instructions for use

First use: Inside the plastic dispenser is roll of wipes in a plastic package. Open the dispenser and cut an opening to the plastic packaging of the roll. Pull a wipe through the distribution hole in the inner cover.

Tear off the wipe and close the packaging lid. Wipe the contaminated area and let it dry. The surface may be dried if required after one minute. Dispose the used wipe appropriately.

The most sensitive surfaces are recommended to be wiped with water or with a cloth humidified in a mild soap solution.

Note

Always close the lid of packaging after use. An open package is to be used within 3 months. Refill package (8085) available.

Composition

Ingredient	Effect
Didecylmethylammonium chloride	Biocide
Alkylbenzyl dimethylammonium chloride	Biocide
Alkylethylbenzylmethylammonium chloride	Biocide

Product pH

7.5 neutral

Storage	Shelf life is printed in the wipe package (inside the dispenser). < +28°C, shelf life 3 years from manufacturing.		
Appearance and scent	Mild scent, product is unscented.		
Country of manufacture	GERMANY		
User and environmental safety	Serious incidents in relation to Cleanisept Wipes are to be reported to asiakaspalvelu@kiilto.com and the local medical device authority.		
Package size / Product code	Sales unit	Product Code	GTIN
	12 x 1 pcs	8080	4260068629711
	12 x 1 pcs	8085	4260068629704
Additional information	Check the updated version of a paper brochure on our website at www.kiilto.com/en or send an e-mail to our customer service at asiakaspalvelu@kiilto.com.		